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Dear Customer,

thank you for choosing a Bauerfeind product that meets very high medical and quality standards. You have purchased a custom-made foot orthosis that was individually milled according to your specifications. This is not a ready-to-use medical product for orthotic care, but rather an orthotic blank. This is a component used to manufacture a custom-made product.

Uses / Indications

Milled orthotic blanks are used to manufacture orthopedic aids for the prevention and conservative treatment of foot deformities.

Please note the following modification instructions:

- The orthotic blank must be modified by trained specialists* according to the consumer's needs (shape, fit, material thickness) for a proper fit that matches the indications.
- We recommend conducting a standing and walking test before delivery to the consumer.
- When making modifications, follow the fit of the consumer's shoes (see Figure a, b and c).
- Always check the suitability of the shoes (for care associated with specific types of sports, only use in the relevant shoes for that sport!).
- A suitable top cover must always be applied on the top side of the foot orthosis.
- Instruct the consumer how to use and clean the aid.
- Provide the customer instructions for use and a declaration of conformity.

Material

Polyester (PES), Polyurethane (PUR)

Version: 2020-04

* A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic foot orthoses.

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