 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-35 00
F +49 (0) 36628 66-39 99
E info@bauerfeind.com

ViscoHeel® K

Viskoelastische Einlagen
Viscoelastic heel cushions
Talonnettes visco-élastiques
Taloneras viscoelásticas

de	deutsch	2	mk	македонски	24
en	english	3	sr	srpski	25
fr	français	5	sl	slovenski	27
nl	nederlands	6	ro	romanian	28
it	italiano	7	el	ελληνικά	30
es	español	9	tr	türkçe	31
pt	português	10	ru	русский	33
sv	svenska	12	et	eesti	34
no	norsk	13	lv	latviešu	35
fi	suomi	14	li	lietuvių	37
da	dansk	16	uk	українська	38
pl	polski	17	he	עברית	41
cs	česky	19	ja	日本語	41
sk	slovensky	20	ko	한국어	43
hu	magyar	21	ar	عربي	45
hr	hrvatski	23	zh	中文	46

de deutsch

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en english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoHeel K is a medical device for the feet. It is a viscoelastic heel cushion It is used as a supination or pronation cushion. ViscoHeel K can be used to strategically correct varus and valgus misalignments of the heel bone as well as misalignments of the leg axis.

Indications

- For varus and valgus misalignment of the heel

Risks of using this product

⚠️ Caution*

ViscoHeel K must only be worn in accordance with the specifications in these instructions for use. To ensure the optimum fit of the ViscoHeel K, the size required must be determined exactly. Only a trained specialist¹ may carry out the (initial) fitting of the product and provide instruction.

Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.

Application instructions

Inserting the ViscoHeel K

Use the ViscoHeel K in shoes with a stiff heel counter. Discuss the use of the ViscoHeel K with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

The ViscoHeel K heel cushions are used to correct varus or valgus misalignments of the heel. The heel cushions are marked with the letters A and B.

- To correct a varus misalignment of the heel, insert the heel cushions as follows: heel cushion A in the right shoe, heel cushion B in the left shoe.
- To correct a valgus misalignment of the heel, insert the heel cushions as follows: heel cushion A in the left shoe, heel cushion B in the right shoe.

Unless otherwise prescribed by your physician, place **both** ViscoHeel K heel cushions in your shoes so they fit snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

Cleaning instructions

Note: Never expose the ViscoHeel K to direct heat (e.g. heaters, sunlight, in your car, etc.!). This may cause damage to the material, which can impair the effectiveness of the ViscoHeel K.

If necessary, you can clean your ViscoHeel K heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

Notes on reuse

This product is intended for your treatment only.

Places of use

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoHeel K have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2020-12

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the products).

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.