**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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en english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

The GenuLoc is a medical device. It is an orthosis for immobilizing the knee joint – either with the leg extended or at a preset flexion angle.

 **GenuLoc must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis<sup>1</sup>.**

## Indications

- Pre-operative, post-operative, post-traumatic immobilization
- Non-surgical treatment of patellar fractures

## Risks of using this product

### Important Information

- Hypersensitive reactions harmful to health have not been reported to date. An immobilizing knee splint is usually prescribed

by a doctor. If this does not apply in your case, you should consult your doctor before fitting and wearing this device if you suffer from the following conditions:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins.
- Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

- GenuLoc is a prescribed product that should be worn under a physician’s guidance. GenuLoc should only be worn in accordance with the instructions contained in these usage directions and for the listed areas of application.
- No liability is accepted for improper use of the product.
- Improper changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your doctor.
- Avoid the use of ointments, lotions or any substances including grease or acids.

## Contra-indications

Side effects, involving the body as a whole, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

## Procedure for first fitting of GenuLoc

Fitting the brace for the first time:

- Before the system is fitted, the circumference setting for the brace width should be adjusted via the lateral Velcro strips. The side sections must be fastened symmetrically (Figures 1 / 2).
  - When positioning the system against the leg, ensure that the central straps are equidistant from the centre of the patella (Figures 3 / 4).
  - To fasten the system, start with the lower strap nearest the joint (Figure 5).
  - Next, fasten the upper strap nearest the joint (Figure 6).
  - Fitting of the brace is completed by fastening the lower strap furthest from the joint and then the upper strap furthest from the joint (Figures 7 / 8).
  - To secure the orthosis definitively, start by uniformly tightening the two lower opposing straps. Repeat the procedure for the two upper straps (Figure 9).
  - If a pad needs to be inserted at the back of the knee (ask your doctor about this), fasten the pad to the Velcro section provided on the inner aspect of the brace before fitting the brace.
- When removing GenuLoc, undo the straps in the reverse sequence to facilitate leg withdrawal.**
- Since GenuLoc is supplied in one universal size that can be adjusted to the individual, no size selection is involved.

## Cleaning instructions

- Please only wash the orthosis by hand at 30 °C using a mild detergent. All components can stay on the orthosis during washing. Leave the orthosis to air dry.
- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the label underneath the removal aid.
- Never expose the product to direct heat as this may impair its effectiveness.
- If you have any complaints, please contact your retailer. Only clean goods are handled.

## Part of the body this product is used for

In accordance with the indications (knee). See purpose.

## Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

## Technical specifications / parameters

The GenuLoc is a stabilizing orthosis for the knee joint. It consists of supporting material, 4 stabilizing stays and 4 straps.

## Notes on reuse

The GenuLoc is intended for your treatment only. It has been adjusted specifically for you. For this reason, do not pass the GenuLoc on to other people.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyamide (PA), Aluminium (AlMgSi1), Polyester (PES), Polyoxymethylene (POM), Polyurethane (PUR), Cotton (CO), Spandex (EL), Polyacrylate (PAK), Viscose (CV)

 – Medical Device  
 – DataMatrix Unique Device Identifier UDI

Version: 2021-01

<sup>1</sup> Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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