AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MalleoLoc L is a medical device. It is an orthosis for the lateral stabilization of the ankle.

⚠ During the first fitting of the MalleoLoc L, the support must be individually adapted to the patient by a trained specialist². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis.

Indications

- Non-surgical treatment of sprains and ruptured ligaments
- Acute and chronic capsular ligament instabilities
- Prevention of chronic ligament instability
- Post-operative rehabilitation / protection, e.g. after suture repair / ligament reconstruction
- Post-traumatic / post-operative irritation

Risks of using this product

⚠ Caution³

- The MalleoLoc L must only be worn in accordance with the specifications contained in these instructions for use, and the areas of application listed.
- The use in combination with other products, e.g. when wearing compression stockings as part of compression therapy, must be discussed beforehand with your physician.
- If you notice any unusual changes or your symptoms get worse, stop using the MalleoLoc L and **please** contact your physician without delay.
- No product liability is accepted in the event of improper use.
- Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thus canceling any product liability.
- Side effects involving the body as a whole have not been reported to date.
- Correct usage / fitting is assumed. Do not fit the MalleoLoc L too tightly, as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the MalleoLoc L.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the part of the body being treated, particularly if inflammation is present. Likewise, any raised

scars with swelling, redness, and excessive heat build-up.

- Impaired sensation and movement disorders of the legs / feet (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn.

Application instructions

Putting on the MalleoLoc L

We recommend that hosiery should be worn under the MalleoLoc L.

1. Sit down to put on the MalleoLoc L.
2. Loosen the upper and lower strap. The lower strap remains threaded into the slot on the sole. Loosen the lower strap so far that you can easily put your foot through the loop you created. **1**
3. Place your foot on the floor at a 90° angle to your lower leg, as this is the best position for fitting the orthosis. Now slide the affected foot into the orthosis between the strap loop and the shell, so that the shell sits laterally on the foot **in front of** the lateral malleolus.
4. Using one hand, press the shell onto the foot so that it sits flat against it. **2** Use the other hand to guide the lower strap diagonally across the instep by pulling from the outside of the foot (from the shell). Fasten the strap at the lower Velcro triangle **3**. During this process, the side of the strap with the blue velour must be attached to the blue plastic shell.
5. Now wind the upper strap around the base of the calf by pulling the strap slightly. **4** then fasten it. **5**
6. Now guide the lower strap above the medial malleolus and around the back of the Achilles tendon, **6** returning it to the front of the foot of the affected ankle. Fasten the end of the strap on the strap (the Velcro area depends on the calf circumference). **7**
7. Tighten both straps again, if necessary. **8** In the event of increased sensitivity on the sole of the foot, you have the option of removing the Velcro pad.

Taking off the MalleoLoc L

1. Open the two Velcro straps.
2. Remove your foot from the orthosis.

Cleaning instructions

The sewn-in label of the MalleoLoc L, which contains information on the product name, manufacturer, washing instructions, and CE mark, is located on the inside of the upper strap.

Note: Never expose the MalleoLoc L to direct heat! This may damage the material, which can impair the effectiveness of the MalleoLoc L.

To clean the MalleoLoc L, open the Velcro fastenings. Fasten the ends back onto the straps to avoid any damage. Use a damp cloth to wipe the shell. To clean the cushion of the MalleoLoc L, remove it from the shell's Velcro. You can wash it separately by hand with mild detergent.

Part of the body this product is used for

In accordance with the indications (ankle).

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The MalleoLoc L is supplied ready for use.

Assembling the MalleoLoc L after cleaning:

Fitting the cushion:

Attach the cushion to the plastic shell using each Velcro fastening point. Start with the sole. Make sure the cushion always protrudes slightly beyond the shell, and there are no folds.

Fitting the straps:

The example below is for fitting the straps for the right side MalleoLoc L.

Place the plastic shell in front of you with the sole pointing toward you, and the outside pointing up (cushion facing down). Take the upper, shorter strap and position it in front of you with the blue velour pointing up. Thread the transparent Velcro end into the right-hand slot of the upper three slots, and back to the outside through the slot immediately adjacent. Attach it to the strap. Now take the other strap end, guide it toward the left in the direction of the left slot, and push it through the plastic shell from the outside.

Take the lower, longer strap and position it in front of you with the blue velour pointing down. Insert the transparent Velcro end in the slot in the middle of the orthosis from the right-hand side and from above, and attach it to the strap on the right. Take the other strap end and insert it into the opening in the sole.

Technical specifications / parameters

The MalleoLoc L is an ankle orthosis with two straps attached to it. The straps and the cushion can be removed.

Notes on reuse

The MalleoLoc L is intended for your treatment only. For this reason, do not pass the MalleoLoc L on to other people.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoLoc L have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyketone (PK), Polyamide (PA), Polyurethane (PUR), Cotton (CO), Polyester (PES), Spandex (EL)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-08

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

² A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

³ Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

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