AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Barrierefreie Version

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AlMg3, Polyamid (PA), Polyurethan (PUR), Polyethylen (PE), Polyoxymethylen (POM), Polyester (PES)

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

The ManuLoc long Plus is a medical device. It is an orthosis¹ for immobilizing the wrist, parts of the forearm and the fingers with a removable finger support.

If the product is not used properly, any claim for warranty will expire.

Indications

- Post-traumatic (multiple, complex injuries in the hand and fingers)
- Pre-operative and post-operative care, e.g. Dupuytren's contracture, metacarpal fractures
- Carpal tunnel syndrome (severe – support at night)
- Tenosynovitis (severe)
- Sprains / contusions
- Ligament injuries in the finger / hand area

Risks of using this product

Caution²

- The ManuLoc long Plus must only be worn in accordance with the specifications contained in these instructions for use and the areas of application listed.
- To ensure the optimum fit of the ManuLoc long Plus, the size of ManuLoc long Plus required must be determined by a trained specialist³.
- Information on the size, manufacturer, washing instructions and CE mark is located on the sewn-in label in the insertion opening for the wide stay located on the inside of the hand (the volar stay). Only a trained specialist may conduct the (initial) fitting of the orthosis and provide instruction.
- Use in combination with other products, e.g. as part of compression treatment, must be discussed beforehand with your physician.
- Should you notice any unusual changes or experience discomfort, please contact your physician without delay.

- No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed. Any medical aids – such as supports and orthoses – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict the underlying blood vessels or nerves.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and circulatory disorders of the arm / hand, e.g. in the case of diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the ManuLoc long Plus

- ➊ Start by loosening the four Velcro straps on the forearm splint and the
- ➋ Velcro strap on the finger support.
- ➌ Place the ManuLoc long Plus on the affected hand. The straps should be located on the top of the forearm and on the back of the hand. First, close the wide strap in the wrist area, then move up the arm, fastening the straps in order. Fasten the narrow strap on the back of the hand.
- ➍ Then loosen one side of the finger strap, wrapping it around the finger joints and
- ➎ velcroing it to the underside of the finger support.

Taking off the ManuLoc long Plus

Undo the straps on the lower arm splint, starting at the finger support and finishing at the lower arm. Take off the ManuLoc long Plus.

Cleaning instructions

Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
- Please only wash the orthosis by hand at 30 °C / 86 °F (cold wash) using a mild detergent. All components can stay on the orthosis during washing. Leave the orthosis to air dry.
- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the label underneath the removal aid.
- Never expose the product to direct heat as this may impair its effectiveness. If you

have any complaints, please contact your retailer. Only clean goods are handled.

Part of the body this product is used for

In accordance with the indications (wrist, lower arm, fingers). See intended use.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical specifications / parameters

The ManuLoc long Plus is an anatomically contoured orthosis. It immobilizes the entire hand and parts of the forearm and helps prevent undesired movements. Two lateral stays and an adaptable metacarpal stay stabilize the forearm and metacarpus. The adjustable finger support immobilizes all four fingers. Large Velcro fastenings fix the orthosis on the arm. The straps on the lower arm splint are detachable. The straps and the finger support are available as spare parts.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc long Plus have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

AlMg3, Polyamide (PA), Polyurethane (PUR), Polyethylene (PE), Polyoxymethylene (POM), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-08

Specialists

General instructions

- Check the optimal and individual fit of the ManuLoc long Plus at the first fitting. Make any corrections if necessary.
- Practice the correct procedure for putting on the support with the patient.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, thereby voiding any product liability.
- The ManuLoc long Plus is intended for the treatment of one patient only. The aluminum stays are anatomically contoured, but they can still be individually adjusted to the patient.
- The finger support is supplied in a neutral position and can be bent into the required position (e.g. the intrinsic plus position) depending on the treatment (at the height of the metacarpophalangeal joints).
- In the case of an assembled orthosis, adapt the dorsal extension angle in the wrist and the palmar flexion angle in the metacarpophalangeal joints. To do this, form the required support with your fingers in order to define the bending edge.

Shaping the stays:

- ➊ Carefully remove the stays from the insertion pockets and shape them to the anatomy of the patient. Reinsert the three stays into the pockets provided.
- ➋ Loosen the four Velcro straps on the forearm splint and the Velcro strap on the finger support. The proceed as described in Section 3.
- ➌ The adjustable finger support immobilizes all fingers except for the thumb. Large Velcro fastenings fix the orthosis on the arm.

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

² Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

³ A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses

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