AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Purpose

The GenuTrain P3 is a medical device. It is a support for centering and securing the patella with an adjustable corrective strap.

Indications

- Irritation of the knee joint (patellofemoral pain syndrome, patellar chondropathy, chondromalacia, patella tip syndrome / jumper's knee, runner's knee)
- Patellar lateralization
- High-riding patella (patella alta)
- After lateral release surgery
- Feeling of instability

Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the GenuTrain P3. They will also inform you whether the GenuTrain P3 can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the GenuTrain P3 as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The effects of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.

- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the GenuTrain P3 – e.g. an increase in discomfort – stop using it and contact your physician immediately.
- The GenuTrain P3 active support cannot mechanically prevent the kneecap from dislocating again (patellar luxation) as a result of external trauma.

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars, swelling, redness, and excessive heat build-up
- Varicose veins (varicosis)
- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in the case of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Fitting

1 Grip either side of the support at the top ends of the lateral spiral stays **2** and pull the support over the knee **3** until the kneecap is enclosed by the massage pad without any pressure being exerted on it. **4** Adjust as required until a snug fit is achieved.

Securing the kneecap in the desired position

5 To ensure that the kneecap is in the optimal position, bend the leg slightly (max. 20°). Release the tensioning strap from the Velcro area on the inside of the support. Pull the strap diagonally upward until the kneecap is in the desired, comfortable position. **6** Refasten the ends of the strap onto the Velcro area.

Removal

7 To remove the support, pull down at the bottom of the support on both sides near the stays. To avoid damaging the support, do not turn it inside out!

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat/cold. Also observe the instructions on the sewn-in label on the upper edge of your product.

Please note the following:

Secure the end of the strap in the indicated position.

Regular care guarantees optimum effectiveness of the product.

Where to apply the product

see Indications and Intended purpose

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The product is supplied ready for use

Technical specifications / parameters

The GenuTrain P3 consists of a knitted section with an integrated massage pad around the knee, a corrective strap on top of this and a massage pad at the upper edge of the support. Spare parts are not available.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain P3 have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Spring steel wire, galvanized, Elastodiene (ED), Silicone (SI), Polyurethane (PUR), Spandex (EL), Viskose (CV), Cotton (CO), Polyethylene (PE), Polyester (PES)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

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