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Dear customer,
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

LumboTrain / LumboTrain Lady are medical products. The supports provide support for the back muscles for discomfort in the lumbar spine region.

Indications

- General pain in the lumbar spine region (acute and chronic)
- Degenerative conditions of the lumbar spine (e.g. osteochondrosis, osteoarthritis, lumbar deformities, spondylolysis without vertebral displacement)
- Status post prolapse (non-surgical / post-operative)
- Status post discectomy

Risks of using this product

- The benefits of this product are felt particularly during physical activity.
- Please remove the support during longer periods of rest.
 - Once you have been prescribed the LumboTrain / LumboTrain Lady, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
 - Avoid contact with ointments, lotions, or substances containing grease or acids.
 - Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
 - If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is applied during intense physical exertion).

Application instructions

Applying the LumboTrain / LumboTrain Lady

- Fasten the pad to the center of the Velcro surface of the support so that the opening for the silicone dots runs vertically down the center. 
- Place your fingers in the hand straps and the support around the waist.  The pad must sit in the middle of the back.
- Pull the two halves of the fastening forward evenly and guide them one over the other: press the left side against the stomach and attach the right side. When fastening the support, remove your fingers from the left hand strap.  

Cleaning instructions

Please use mild detergent and a lingerie bag. Remove the pad and wash it by hand. Close all the Velcro fastenings. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Where to apply the product

See Indications and Intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is maintenance-free.

Assembly and fitting instructions

The product is supplied ready for use.

Technical specifications / parameters

- Your product comprises:
- Support with abdominal fastening
 - Dotted pad.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the LumboTrain / LumboTrain Lady have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the local health authorities. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Thermoplastic elastomer (TPE), Polyamide (PA), Thermoplastic polyurethane (TPU), Spandex (EL), Polypropylene (PP), Polypropylene copolymer (PPC), Polyurethane (PUR), Cotton (CO), Polyester (PES), Polydimethylsiloxane polyurea block copolymer (PDHC), Polyethylene (PE), Copolyester (TPC)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.