**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@Bauerfeind.at

**BENELUX**

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(en) english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended use

ManuTrain is a medical device. It is a support for the relief and stabilization of the wrist.

**⚠ ManuTrain must be fitted for the first use by a trained specialist\*, otherwise we cannot accept any liability for the product.**

## Indications

- Chronic, post-traumatic or post-operative irritation
- Distortions (wrist injuries e.g. sprains)
- Osteoarthritis (joint wear)
- Instability, e.g. radioulnar subluxation
- Tendovaginitis (tenosynovitis)
- (Relapse) prevention

## Risks of using this product

### ⚠ Important Information

- ManuTrain is a prescribed product that should be applied under a physician's guidance. ManuTrain should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product should not undergo any improper modification. Failure to comply with the requirement can impair product performance and thus rule out any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- A combination with our products must be discussed with your medical advisor beforehand.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

## Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
  2. Impaired sensation and circulatory disorders of the hand.
  3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports or orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

## Application instructions

### Fitting ManuTrain

1. Open the Velcro® fastening.

2. Slip the support down the hand so that the wrist is enclosed, pressure free, in the profile inserts ①.
3. The support is positioned correctly if the ulnar condyle is enclosed by the small oval cut-out ②.
4. Finally close the Velcro® fastening.

### Taking off the support

- Undo the Velcro strap and take off the support ③.

### To modify the thermoplastic stay

The inner hand stay can be removed from the support for individual adjustment. To do this, pull the inner hand stay out of the pocket provided on the inside of the support ④. Heat with a hot air gun at 180 °C (365 °F) as required.

### Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

- For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.

### Notes on reuse

This product is intended for your treatment only.

### Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Version: 2019-09

\* A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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