**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E [info@bauerfeind.at](mailto:info@bauerfeind.at)

**BENELUX**

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product. Please read the **instructions for use** carefully and contact your physician or medical retailer if you have any questions.

## Intended purpose

MyoTrain is a medical device. It is a support with adjustable compression for the treatment of thigh muscle injuries.

## Indications

- Muscle hardening
- Muscle strains
- Muscle bruising (thigh knock)
- Muscle fiber tears
- Tears in bundles of muscle fiber, for example the rear thigh muscles (hamstrings)
- Injuries to the fascia lata
- Post-traumatic, post-operative irritation
- Prevention / recurrence prevention

## Risks of using this product

- MyoTrain should only be worn in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The use and duration of use must be discussed with the physician who prescribed the MyoTrain. They will also inform you whether the MyoTrain can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the MyoTrain as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The effects of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to your use of the MyoTrain – e.g. an increase in discomfort – stop using it and contact your physician without delay.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

## Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Varicose veins
- Impaired sensation in the leg due to neurological abnormalities
- Impaired circulation in the leg (e.g. diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

## Application instructions

### Putting on the orthosis

1 Put on the MyoTrain standing up, with a relaxed leg, and a slight bend in your knee. Put the support on, making sure that the rear Velcro straps (outer) are positioned in the middle of the injured muscle area. The entire injured area must be covered by the support.

2 Now roll the upper edge of the support distally (downward) approx. two thirds of its length.

3 Place the large pad (functional pad) on the affected area of the thigh. The smaller pad (acute pad) can be inserted if you would like to increase the pressure. Next, roll the support back up again, making sure that there are as few creases as possible.

4 Attach the Velcro straps to the Velcro strip on the back. The straps must be guided from bottom of the rear to the top of the front. The short straps are at the bottom and the long ones at the top.

5 The tensioning system (eyelets) is to be attached to the Velcro straps at the front. It is important to ensure that the straps are guided smoothly through the eyelets. 6 Starting with the lower pair of straps, tighten and fasten the inner and outer straps at the same time. The pressure of the support and the straps should be gently adjusted when they knee is bent so that the patient feels the regulated compression but this is not painful.

Once the basic fitting has been performed correctly as described above, the support

can subsequently be fitted and removed easily simply by opening and closing the straps. Be careful to ensure that the basic positioning for the course of the straps is not modified.

## Use of the pad

The following sketches show you the various options for inserting the cushions depending on the stage of treatment:

1. Immediately after the injury:

Locate the acute pad directly over the injured site. Position the functional pad on top of the acute pad. Both pads can be pre-cooled. The acute pad can be removed after approx. 24 hours (Fig. 3). After this time the increased compression produced by the acute pad is no longer required.

2. Subsequently:

You can continue to wear the larger functional pad, particularly when resuming training. 7 You can attach the side wings of the functional pad to both sides as required.

3. MyoTrain can be worn without pads.

## Cleaning instructions

**Note:** Never expose the support to direct heat (e.g. heaters, sunlight, leaving in your car, etc.) This may cause damage to the material, which can impair the effectiveness of the support

Please refer to the care instructions on the sewn-in label. This can be found on the upper edge of the support.

We recommend hand-washing the MyoTrain separately at 30 °C with a liquid detergent (not a heavy-duty detergent). Please note the following:

1. Remove the strap components from the fabric and wash the support on its own.
2. The strap components and pads can also be washed on their own by hand as required.
3. MyoTrain is not suitable for tumble drying.

To make a complaint, please contact the retailer that provided you with the product.

**Note:** Only clean goods are handled.

## Notes on reuse

This product is intended for your treatment only. Do not pass the support on to other people.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MyoTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyamide (PA), Iron with polyamide (Fe with PA), Elastodiène (ED), Polyurethane (PUR), Silicone (SI), Viscose (CV), Polyester (PES)

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