deutsch

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english

## Dear technician,

thank you for choosing a Bauerfeind product that meets very high medical and quality standards. You have purchased a custom-made foot orthosis that was individually milled according to your specifications. This is not a ready-to-use medical product for orthotic care, but rather an orthotic blank. This is a component used to manufacture a custom-made product.

### Uses / Indications

Milled orthotic blanks are used to manufacture orthopedic aids for the prevention and conservative treatment of foot deformities.

### Main indications

- Flat foot/splay foot
- Skew foot

### Concomitant indications

- High-arched foot, mild and moderate severity<sup>1</sup>
- Post-traumatic foot deformities<sup>1</sup>
- Chronic heel and ankle pain<sup>1, 2</sup>
- Achilles tendon complaints<sup>1</sup>
- Knee joint/back pain<sup>1</sup>

### Please note the following modification instructions:

- The orthotic blank must be modified by trained specialists\* according to the consumer’s needs (shape, fit, material thickness) for a proper fit that matches the indications.
- We recommend conducting a standing and walking test before delivery to the consumer.
- When making modifications, follow the fit of the consumer’s shoes (see Figure a, b and c).
- Always check the suitability of the shoes (for care associated with specific types of sports, only use in the relevant shoes for that sport!).
- A suitable top cover must always be applied on the top side of the foot orthosis.
- Instruct the consumer how to use and clean the aid.
- Provide the customer instructions for use and a declaration of conformity.

### Application instructions

- The milled orthosis must be fitted to the feet and/or shoes by a qualified specialist<sup>1</sup>.
- When selecting the correct size of foot orthoses, the packaging information serves only as a guide. Just like correctly fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot when the foot is in a weight-bearing, standing position.
- The shoes must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length/width, removable, sufficiently thick cushioned sole, adequate lateral guidance of the foot orthoses in the case of open shoe types). An uncomfortable fit that constricts the foot must always be avoided.
- The forefoot (a1), midfoot (a2) and hindfoot (a2) must be supported sufficiently evenly from below by the foot orthosis and must not be positioned on one of its edges.
- Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your physician.
- It must always be ensured that the left and right foot orthoses are clearly identified in a way which is easily understood by the patient.
- The foot orthoses must not be exposed to temperatures exceeding 60 °C (e.g. heating, hair-dryer, direct sunlight).
- The materials used for the foot orthoses are subject to aging and their properties can therefore change.

### Storage conditions

The products must not be exposed to direct UV light and must be stored within a temperature range of 10 °C to 25 °C and free from any mechanical influence. The manufacturer recommends that regular checks are carried out on both the foot orthoses in storage and the storage conditions.

### Material

Polyester (PES), Polyurethane (PUR)

Version: 2023-07

\* A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic foot orthoses.

<sup>1</sup> Depending on the severity of the foot, knee, or back deformity

<sup>2</sup> Depending on the structural design of the foot orthosis or variant