AUSTRIA
Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0)800 4430-130
F +43 (0)800 4430-131
E info@bauerfeind.at

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en english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear Customer,

Thank you for choosing a product from Bauerfeind. GloboPed is a high-quality medical product. Please read through these **instructions for use** carefully and if you have any questions contact your physician, medical retailer, or our Technical Service department.

Product Description

The **Aktiv strap shoe** is a prescribable product that may only be worn under a physician's direction and only in accordance with these instructions of use and worn for the applications named.

Purpose

The **Aktiv strap shoe** is a high-quality orthopedic device for early mobilization of a bandaged foot.

Indications / Use

- For early post-operation and / or post-traumatic mobilization of the foot and for wound bandages on the foot
- Treatment for foot deformities and inflammation

Contraindications / Application Risks

Mobility may be compromised based on the purpose and desired function of the **strap shoe**. There are no known side effects for the entire body. The **strap shoe** should only be worn after consulting trained technical personnel¹. There are no known hypersensitivities that cause illnesses. This device should only be worn after consulting your physician in the following clinical situations:

- Skin diseases / injuries in the body part being treated, particularly if inflammation appears. The same applies for accumulated scars with inflammation, redness, or overheating
- Acroparesthesia in the body part being treated
- Lymphoid circulation problems, unclear soft tissue inflammation around the shoe's area of application

Application Instructions

The **strap shoe** should be put on while sitting and is available both on the left and right foot. Put on with the Velcro snugly on the foot, although not too tight in order to avoid pressure points on the skin respectively damages of arteries and nerves. An individual orthopedic foot insert may be worn with the **Aktiv strap shoe** in place of the inner sole delivered with shoe. The insert may not negatively affect the inner shoe volume available to the foot. Always wear a non-skid shoe with the same sole height on your healthy foot. If necessary, a physician may prescribe that the height of the sole be changed by an orthopedic technician or orthopedic shoe technician. Any open wounds on the foot must be protected by a bandage or similar device. Do not do not ride a bicycle or drive a car or any other vehicle while wearing the shoe. Use crutches when wearing a **strap shoe**. Wearing the **Aktiv strap shoe** is only recommended in hospitals and clinics or within your own home and not in public areas. Please see your physician immediately should you notice any irregular changes in your body (i.e. increase in pain). Be sure to have your **Aktiv strap shoe** checked regularly for functionality and closing functionality by an orthopedic technician or orthopedic shoe technician and exchanged if needed.

GloboPed®

Verbandschuh »Aktiv«
"Aktiv" strap shoe
Chaussure couvre-pansement « Aktiv »
Zapato para vendas elásticas »Aktiv«

All instructions for use:
www.bauerfeind.com/downloads

Combining the aid with other medical products (e.g. compression stockings, supports) may only be considered after having sought specialist advice. We have tested this product within the scope of our integrated quality management system. Should you nevertheless have a complaint, please contact our local technical consultant or our customer service department.

Cleaning Instructions

We recommend washing the strap separately at 30 °C and by hand using a delicate detergent. We recommend closing the Velcro before washing in order to preserve its functionality and avoid damage. The orthosis should not be exposed to direct heat when drying (i.e. heating vents, sun light, etc.).

Technical Information

Model	Size	Article Number left	Article Number right
Aktiv	35–36	3 662902 3600000	3 662901 3600000
	37–38	3 662902 3800000	3 662901 3800000
	39–40	3 662902 4000000	3 662901 4000000
	41–42	3 662902 4200000	3 662901 4200000
	43–44	3 662902 4400000	3 662901 4400000
	45–46	3 662902 4600000	3 662901 4600000
	47–48	3 662902 4800000	3 662901 4800000

Notes on reuse

This product is intended for the treatment of one individual patient and may not be reused or used by others.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists¹. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty. First-time application and fitting of the **Aktiv strap shoe** may only be carried out by trained specialists¹, otherwise product liability cannot be assumed.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2016-08

¹ A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic aids.

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